EMO
especialistas
en ortopedia

Ortec walkers

Dear customer, thank you for the trust placed in EMO product. Picking a brace of Ortec line you have acquired a quality product and high health medical level. Read these instructions carefully. If in doubt contact your doctor or specialist dealer.

NORMATIVE

Regulations The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND CHARACTERISTICS

Security during ambulation: slip sole seesaw High stability: wide side decks. Maximum comfort: breathable cover and fully padded booty to cushion impacts during ambulation. Velcro closure system by producing a homogeneous stability, providing stability and comfort. Emo-Walker W100 Short: tibial orthosis immobilizing the ankle at 90 degrees. Short model. Emo-Walker W110 High: tibial orthosis immobilizing the ankle at 90 degrees. High model can cover the entire length of the tibia. Emo-Walker W115 High Tire: Bracing tibial immobilizing the ankle at 90 degrees. Perfect fit: adjustable pneumatic cushioning adjustment. Shinguard that distributes pressures. High model can cover the entire length of the tibia. Emo-Walker W210 Alto joint: Bracing tibial immobilizing the ankle joint. Adjustable joint: every 7,5° in any position between 45° of dorsiflexion and plantar flexion 45. High model can cover the entire length of the tibia. Emo-Walker W215 High Tire with joint: Bracing tibial immobilizing the ankle joint. Perfect fit: adjustable pneumatic cushioning adjustment. Adjustable joint: every 7,5° in any position between 45° of dorsiflexion and plantar flexion 45. Shinguard that distributes pressures. High model can cover the entire length of the tibia. W315: Bracing immobilizing tibiotaarsiana to 90°. Homogeneously envelops the entire length of the foot, ankle and tibia. Completement booty padding that absorbs shocks during ambulation. Breathable padded adjustable pneumatic adjustment of a wide airlock. Lightweight design bivalvo very stable, durable and anatomical conformation. Maximum stability and comfort. Slip sole shaped rocker facilitates full gear. Double adjustment: clamping webbing and padding tire. It allows the incorporation of

wedges for post-surgical treatment of Achilles tendon, alternative treatment to plaster splint. Chocks W900 Walker: Group of 6 hikes that provide up to plantar flexion of 20°. We eliminate progressively increases reduce plantar flexion up to 0°. It incorporates cushioned insole which is fixed on hikes.

INDICATIONS

Sprained ankle ligaments (grades II and III). Very painful chronic plantar fasciitis. Post-trauma. Conservative treatment of injuries of the tarsus, metatarsals and phalanges. stable distal tibia or fibula fracture. Foot and ankle fracture. Post-surgical Achilles tendon. Post-surgical soft tissue, tendons and ligaments. Reduction of edema.

INSTRUCTIONS FOR USE AND PLACEMENT

1. Place the padded brace wrapping the foot and leg so that the heel fits snugly in the back. The pads supplied can be used to fill the empty spaces around the heel, instep, ankle or calf. 2. Place the tab on the back foot before closing the internal padding. 3. Tightly close the velcro end of the padding at the level of the foot and calf. 4. Place the heel inside the boot. Place the strips on the centerlines ankle and remove the plastic covers, so that the Velcro closure is exposed by adjusting the plates on the padding. The plates must conform to the patient's anatomy. 5. Adjust closing with three Velcro straps of both the tibia and foot. Pneumatic system models incorporate a plate of thermoplast material in order to achieve a uniform pressure. These should be placed between the padding and straps adjustment. The pneumatic system regulates the amount of air by manually pressing the pump located in the upper part of the orthosis to the desired pressure. To this should ensure that the valve is in the position air inlet. To decrease the pressure and thus more easily take your foot / calf place the valve in a position to deflate.

NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If an allergic reaction should occur remove the product and consult with the prescribing physician. In case of minor

annoyances caused by excessive sweating we recommend the use of a tubular cotton. Positioned such that compression is not excessive. Do not expose the garment to a flame or heat source at very high temperatures. Store at room temperature, preferably in the original packaging. Guarantee within the framework of laws.

WASHING AND MAINTENANCE

Hand wash in warm water (maximum 30 °) and mild soap. During washing, if the garment has Velcro, they must be glued together or keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may damage the garment.

QUALITY CERTIFICATES

All materials used have been tested, comply with applicable European standards are of high quality and offer maximum safety and comfort during use.

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