Dear customer, we appreciate the trust placed in a product EMO. Choosing a brace Ortec line you have purchased a quality product and high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or your specialist.

#### Regulations

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

#### Description and Features

The AFO is a brace antiequina posterior leaflet design polypropylene such as "Rancho of friends" for the treatment of equinus in flaccid paralysis with or without varus deviation. Its variants AF0100 Media Plant Whole Plant AF0200 Varo AF0110 Media Plant Whole Plant AF0210 Varo, antiequinos allow different effects and / or correction of varus deviations associated with it. Sizes (No shoes / height in cm.) Pediatric (25-30/18, 5), S (31-35/28) M (36-40/34), L (41-45/39)

#### Effects

The orthosis opposes a resistance to plantarflexion that avoids foot drop during phases equinus heel support and oscillation. It is indicated in cases of severe spasticities. Varo variant plays a antivarо, adjustable tension normally associated equinus.

#### Directions

Pie equino flaccid paralysis with or without varus deviation.

#### Instructions for use and placement

The product should be adapted by the prosthetist or by the physician on an individual basis for each patient. The right choice of size is essential for maximum treatment efficacy and prolong the life of the garment. For placement of the plantar orthoses enter in footwear (we recommend using a wide and faceted shoes) that is fitted at the rear. In case of

using the whole plant or model AF0210 AF0200 Varo whole plant, and wherein the shoe size is less far from the plant, will need trimming subsequently smoothing the edges with fine sandpaper. When using the model plant or AF0110 AF0100 Media Media Plant Varo, metatarsal support should not match the cut plantar splint. Then simply adjust padded calf area, without choking, placing the tape in the most comfortable for the patient in the tibial area. The upper cut should always be below the fibular head to avoid discomfort during walking. The orthosis can be placed directly on the skin but for convenience is advisable to use a high sock.

#### Precautions

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a health-care professional. If an allergic reaction should occur remove the product and consult with the prescribing physician. In case of minor annoyances caused by excessive sweating we recommend the use of a tubular cotton. Positioned such that compression is not excessive. Do not expose the garment to a flame or heat source at very high temperatures. Store at room temperature, preferably in the original packaging. Guarantee within the framework of laws.

#### Cleaning and maintenance

For pads and adjustment straps: Hand wash with warm water (max. 30 °) and neutral soap. During washing the velcro should be glued together or keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may deteriorate the garment. Do not bleach or dry clean. To lay the garment dry and not exposed directly to a heat source. For the covers of orthoses, wash exterior and interior surfaces with a cloth moistened with alcohol. La correcta elección de la talla

**Quality Certificates**  
All materials used have been tested, comply with applicable European standards, are high quality and offer maximum safety and comfort during use. Oko-Tex Standard 100: textile eco-label awarded after the completion of a catalog of criteria for analysis to ensure the absence of substances harmful to health. Certify the absence of harmful substances in all the materials that is made of a garment.

#### Warranty

Especialidades Médico Ortopédicas s.l., guarantees that all its products, as long as they have not been manipulated or altered, always are within the framework of the legal terms. Products whose characteristics have been altered due to misuse are not included in this warranty.

#### Notice

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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