Upper limb orthosis

Ortesis posturales

en

Dear customer, we appreciate the confidence placed in a product EMO. Choosing a brace Ortec line you have purchased a quality product and high level of physician health. Read these instructions carefully. If in doubt, contact your doctor or your retailer.

Regulations

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Description and Features

ORTEC MN100 Postural hand flat, MN200 Postural hand ORTEC functional ORTEC MN500 MN600 Paralyzing ORTEC wrist and Thomas splint: splint made up in low density polyethylene for a perfect fit. Removable fastening tapes with improved adaptation and more comfortable for the patient. Easily customizable designs made from lightweight materials that enhance the comfort of the patient. Sizes (palmar width in cm.): S (6-7), M (7-8.5) L (8.5-10).

Postural hand ORTEC MN200 functional anatomical design that keeps the whole hand and wrist in a functional position.

ORTEC MN500 Paralyzing wrist: Design with thumb hole and dorsal closure solapa. Tallas (wrist contour cm.): S (14-17) M (17-20) L (20-23).

ORTEC MN600 Thomas splint: It has a dorsal valve with fitting high-Thomas, for the extension of the metacarpophalangeal and interphalangeal joints with medial piece for thumb abduction. The active voltage regulating spring required for the patient. One size fits all. Right and left sides.

Ortec MN300 hand splint

ORTEC MN500 Paralyzing wrist: Remove the tape around the splint, place in the belly of the forearm in the body of the splint, with the wrist in neutral position, inserting the thumb side hole. Adjust the straps in position and length in the palm, wrist and forearm, ensuring that the flap remains inside the body of the splint.

ORTEC MN600 Thomas splint:

adjust the splint for the dorsal adjusted using tapes, directly or plaster. The fitting may move longitudinally. The length of the elastic force acting determines both finger extension and in abduction of the thumb. Adaptation: The wider tape is preferably placed in the area of the wrist to improve the fit, but may be exchanged if necessary.

Components

MN300 / MN315 consist of the following components: malleable aluminum splint, securing straps to control fingers, hand, wrist and forearm and interdigital separator. Accessories: ACM001 Ortec digital inter-separator; padding material. Velcro adjustment. One size fits all.

Warranty

Especialidades Médico Otopédicas s.l., guarantees that all its products, as long as they have not been manipulated or altered, always are within the framework of the legal terms. Products whose characteristics have been altered due to misuse are not included in this warranty.

MN300 / MN315 constan de los

Ortec ACM002 positioning pads kit: thumb and palm area. Velcro adjustment. Sizes: S (5-7), M (7-9), L (9-11) correspond to the size of the splint. Ortec ACM003 tethers kit: thumb, fingers, wrist and forearm.

Effects

Maintaining effective functional positions and realignment of finger joints, wrist and hand due to the possibility of forming the orthosis to the patient's morphology and position indicated by the practitioner. His whole fixing straps get an effective immobilization and excellent control of spasticity.

Directions

Peripheral and central neurological injuries, deformities prevention and control, control spasticities, requiring postoperative positioning fingers, hand and wrist, carpal tunnel syndrome.

Installation instructions

The product should be adapted by the prosthetist or by the physician on an individual basis for each patient.

The right choice of size is essential for maximum treatment efficacy and prolong the life of the garment. Remove the tape around the splint. Once the splint formed as indicated, the patient is placed on the palmar splint hand and ventral forearm and once set to the correct height to help close the splint correctly.

Remove during the night. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy. The adjustments of the orthotic should be made individually. The orthotic should be used for the purpose it is prescribed for. Do not expose the orthotic to a flame, heat source or high temperatures.

Cleaning and maintenance

Hand wash with warm water (max. 30 °) and neutral soap. During washing, if the garment has Velcro, they must stick together and keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents can damage the garment.

First fixing fingers, wrist and then finally placed and fixed the third strap on the forearm. In the case of the MN315 also be placed on the fourth thumb strap. In the case of placing the inter-digital webbing, it must be fixed to the ferrule before hand on this place.

Notice

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

Precautions

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. The used material is hypoallergenic but if there is any allergic reaction or irritation remove the product and consult with the prescribing doctor.

The user should position itself in such a way that the compression is not excessive. In case of doubt about the size, choose the bigger size. Only use during active periods.

Remove during the night. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy. The adjustments of the orthotic should be made individually. The orthotic should be used for the purpose it is prescribed for. Do not expose the orthotic to a flame, heat source or high temperatures.

ORTEC MN100 Postural hand flat: palmar born ribbon between finger and thumb, fingers setting is introduced into the side slot and runs through the area near the wrist, setting the palm. Postural ORTEC MN200 functional hand-held: the splint with the wrist in neutral position and functional fingers on the plastic zone and the thumb in opposition, as suggested by the splint, adjusting the tape at that level. Adjust the straps in position and length in the wrist and forearm.

ORTEC MN500 Paralyzing wrist: Remove the tape around the splint, place in the belly of the forearm in the body of the splint, with the wrist in neutral position, inserting the thumb side hole. Adjust the straps in position and length in the palm, wrist and forearm, ensuring that the flap remains inside the body of the splint.

ORTEC MN600 Thomas splint: adjust the splint for the dorsal adjusted using tapes, directly or plaster. The fitting may move longitudinally. The length of the elastic force acting determines both finger extension and in abduction of the thumb. Adaptation: The wider tape is preferably placed in the area of the wrist to improve the fit, but may be exchanged if necessary.

In the event that the orthosis MN100, MN200 and MN600 not remain perfectly adapted to the hand morphology, the orthopedic technician can proceed to cutting edges and / or its conformation by applying hot air. Products whose characteristics have been altered due to misuse are not included in this warranty.

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