USER
Dear customer, we appreciate the trust placed in a product EMO. Choosing a brace Ortec line you have purchased a quality product and high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or your specialist.

RULES
The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND FEATURES
The femoral head remain centered within the socket, at 60° of abduction and 90° of flexion, thus promoting normal bone development.

The position of flexion, abduction and external rotation increased coverage of the acetabulum on the femoral head; allows greater stability and consistency. HP100 Pavlik harness, chest harness foamed soft tissue. Rafter straps lined with cotton laminated fabric. Extra-flat buckle snap closure. Adjustable Velcro booties. Double HP200 diaper: soft cotton cloth. Fins and adjust with Velcro. Culotte HP300 abduction: Made with soft cotton fabric waterproof. It incorporates a rigid rectangular pillow between the thighs prevents hip adduction. Fits and adjusts with Velcro. HP350 Becker: Cast in plasticote. Easy adaptation to the thighs and buttocks area. Is set and adjusted by crossed straps that pass over the shoulders of the baby. Diaper HP400 of Frejka: Made with a soft cotton cloth. Composed of a metal structure easily adjustable and padded. Incorporates a padded rectangular pad that is placed between the thighs. With buckle closure. Von Rosen HP450: structure in the form of "H" special aluminum pad. Designed to surround the lower ends of the thighs, the central area around the waist. Padded straps over the shoulders. HP500 Cot Splint: made of thermoplastic padding. Lined with soft terry cloth. Adjusts and adapts Culqui Velcro on the part of the pads.

DIRECTIONS
HP100 Pavlik harness. Diaper HP400 of Frejka: Hip dysplasia, subluxation, or unstable hips luxables in the first months of baby's life, congenital hip dislocation when there is a degree of instability very important, always having achieved a degree appropriate reduction alongside other orthopedic or surgical treatments. Double HP200 diaper, bib HP200 abduction, hip dysplasia, subluxation, or unstable hips luxables through the first two months of baby's life, HP350 Becker, Von Rosen HP450: congenital dislocation or subluxation of the hip, complementing other orthopedic or surgical treatments. HP500 Cot Splint: congenital hip dislocation, reduction is not enforced in hip

dysplasia, hips luxables, or unstable during the first months of baby's life, subluxations other orthopedic or surgical treatments. Unforced reduction in hip dysplasia replacing spica plaster and splints with harnesses.

INSTRUCTIONS FOR USE AND PLACEMENT

The product should be adjusted by the prosthetist or the physician individually for each patient. The correct choice of size is essential for maximum treatment efficacy and prolong the life of the garment. HP100 Pavlik harness, place the chest strap at chest level and keep it to that height with straps. Place your feet in booties and secure. Place the previous films, keeping the hip fixed and fixed. Put the tape later to keep the legs open. Double HP200 Diapers: Diaper using the thigh velcro closure. The fabric must embrace the thigh and the abdomen of the baby. Culotte HP300 abduction: center pad on the front of the baby's legs separating the thighs. Secure the diaper with Velcro. HP350 Becker: focus infant on the casting. Pass the tape on the shoulders and attach with buckles. A degree of mobility of the hip. Diaper HP400 of Frejka: center pad on the front of the baby's legs separating the thighs. Adjust the length of the fitting according to the distance between calves tightening the screws on the plates inside expandable padded area. Put the two side pads embracing both thighs by adjusting the straps and fasten with buckles. Spend two tape straps on the shoulders and later set on the front. Von Rosen HP450: Conform to the level of outgoing trunk and rounded thighs, hugging the trunk at waist and hugging her thighs without preventing the bending of the knees. The tapes will be placed crossed on the chest to prevent displacement with the baby's movements. HP500 Cot Splint: Place the baby supine, elevate the feet to slide the tray below. Embracing the thighs with pillows and fix. Adjust the abdominal area.

NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If an allergic reaction should occur remove the product and consult with the prescribing physician. In case of minor annoyances caused by excessive sweating we recommend the use of a tubular cotton. Positioned such that compression is not excessive. Do not expose the garment to a flame or heat source at very high temperatures. Store at room temperature, preferably in the original packaging. Guarantee within the framework of laws.

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en

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WASHING AND MAINTENANCE

Hand wash with warm water (maximum 30 degrees) and mild soap. During the wash, if the garment has Velcro, they must stick together or keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohol and solvents can damage the garment.

QUALITY CERTIFICATES

All materials used have been tested, comply with applicable European standards, are high quality and provide maximum safety and comfort during use. Certificate Oeko-Tex Standard 100: textile eco-label awarded after the completion of a list of criteria of analysis that guarantees the absence of substances harmful to health. Certify the absence of harmful substances in all the materials that is made of a garment.

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