Infinity

en

Dear customer, we thank you for your trust in the products of EMO. By choosing an orthotic of the Infinity line you have acquired a qualitative medical product. Read these instructions carefully. If you have any questions, please contact your doctor or your specialist.

NORMATIVE

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND CHARACTERISTICS

LS401 Infinity Orthosis Lumbosacral semi-rigid and LS402 Infinity Orthosis Lumbosacral semi-rigid with inelastic fabric are light and very breathable. Adjustments could be regulated by the independent pulleys. This achieves maximum power and immobilizes the body with minimum effort.

Easily adaptable to the male and female anatomy: due to the handle and pulley system the orthotic is perfectly adaptable to the hip and waist. It is incorporated with a sacrolumbar plate of thermoplastic. Large loops for easy placement. DermoDry Coolmax Inside: The technical textile accelerates the process of absorbing moistures to keep the skin dry at all time. Large loops for easy placement. CLS410 Infinity Brace Lumbosacral and CLS420 Infinity Brace Lumbosacral High: The rigid shaped immobilization orthotic is designed for and integrated into a LS401 (CLS410) or LS402 (CLS420) lumbosacral brace. The products are made out of low dense moulded polyethylene and are easily adaptable by the orthopaedic. Due to its flexibility it adapts perfectly to the anatomy of each patient. This orthotic covers the sacrolumbar zone to the abdominal wall. It is incorporated with an independent abdominal plate that is fixed by Velcro. The tensions of the lumbosacral orthotic allow the user to regulate the abdominal compression of the lumbar zone. Lined with soft, light weighted and extremely breathable fabric. DLS230 Back support together with LS401 or LS402 and CLS410 or CLS420 orthoses: extend the lumbosacral orthosis and join the support through the microvelos it incorporates. You can choose the height of the orthosis depending on the level to which the velcro attach. Once the piece is assembled place the orthosis to the patient, passing the shoulder straps above the shoulders, passing under the armpits, crossing over the dorsal area and closing over the abdominal area using the end buckles. Once

closed the buckle pull back to adjust. The orthosis can be used without buckles, removing these and gluing the straps through their microvelos on the abdominal area. Applicable to all orthoses: to conform the steel straps adapting them to the morphology of the patient.

EFFECTS

LS401 Infinity and LS402 Infinity: Stabilization, Immobilization, Intervertebral discharge and Intra-abdominal compression. CLS410 Infinity Brace Lumbosacral and CLS420 Infinity Brace Lumbosacral High: Together with an LS401 (CLS410) or LS402 (CLS420) lumbosacral orthosis; intra-abdominal pressure and interdiscal discharges will be provided. Limits the user in its movements, lateral tilt, rotations and immobilizes. DLS230 together with an inflatable orthoses Infinity LS401 or LS402 and CLS410 or CLS420: shoulder retrouplusion.

INDICATIONS

In conjunction with an Infinity LS401 (CLS410) or LS402 (CLS420) lumbosacral brace the following can be reduced: Low back pain, lumbosacrosis, Facet Syndrome, Discal protrusion, Hernia, Spondylolisthesis, Spondylarthritis, degenerative discopathies, degenerative processes. Discal crush vertebral fractures, muscle hypotonia, postoperative posture. It also can be used for rehabilitation. LS401 or LS402 and CLS410 or CLS420 orthoses: Dorsalgias, osteoporosis, spondylarthrosis, disc degeneration, vertebral fracture, muscle hypotonia, traumasms, postural and post-surgical rehabilitation.

INSTRUCTIONS FOR USE AND PLACEMENT

LS401 and LS402: place the orthosis over the sacrolumbar area at the correct height. To confirm the steels to the morphology of the patient, help the fingers to tighten the orthotic over the abdomen. Close the orthotic by using the Velcro once obtained the desired compression. Grab the lateral tensions and pull to regulate the pressure of the orthotic. Finally fix the tensions on the abdominal area. CLS410 Infinity Brace Lumbosacral and CLS420 Infinity Brace Lumbosacral High: Together with a Lumbosacral orthosis LS401 (CLS410) or LS402 (CLS420); after adapting the orthotic to the patient, fix the orthotic to the body and place it on the sacro-lumbar zone at the correct height. Use the pins with your fingers to tighten and adjust

WASHING AND MAINTENANCE
Hand wash with warm water (maximum 30 degrees) and neutral soap. The Velcro on the products must be kept closed during the washing. Dry at room temperature. Do not iron the garment. Contact with ointments, creams, alcohols and solvents may deteriorate the garment. Do not expose the orthotic to a direct heat source by drying. Do not expose to a direct heat source. Dry and store the product at room temperature, preferably in the original package.

Infinity

fr

closed the buckle pull back to adjust. The orthosis can be used without buckles, removing these and gluing the straps through their microvelos on the abdominal area. Applicable to all orthoses: to conform the steel straps adapting them to the morphology of the patient.

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