Dear customer, we appreciate the trust placed in a product EMO. Choosing a brace Ortec line you have purchased a quality product and high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or your specialist.

Regulations

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Description and Features

Ortec PS100 Antidécubitus heel is made of a soft material that decreases the pressure. Prevent sores that occur on the heel and the ankles and the effects of friction on the area. Adjustment is by velcro. Ortec HAL100 Hallux Valgus Splint: Spell used to get right postural correction or slowing the progression of the deviation of the great toe valgus and varus first metatarsal. Made of terry velor and cotton. It incorporates a molded aluminum strip (medial) suitable for bunion and facilitates postural correction. The fit of the brace is made using velcro. Tobi-Stable Ortec TB100, TB110 Ortec Tobi-Stable and TB200 Ortec EMO-Stable Antiesguines ankle stabilizer that achieves excellent stability thanks to the rigid strap incorporating pronation and supination movements prevents or investment. Adjust and close by lacing. Flexible rods containing half-side. Ortec TB300 stabilizing ankle brace is in two leaflets that cover and stabilize the ankle malleolar level. Both pieces are padded with visco-elastic material with memory for enhanced fit and comfort of the patient. Has a part for adjusting the width of the heel and two tapes to adjust the area of the leg. Ortec TB350 with joint stabilizer, is a hinged brace for control and stabilization of the ankle joint. The adjustment is made by two previous films and later joining the two valves in the leg. The joint is ready for the technician to lock in a certain position. Ortec TB600 ankle immobilizing polyethylene, is a bracing kind of foot on a full enclosure to hug the ankle joint, immobilize and stabilize at 90 °, with relief in the area of malleoli.

Directions

Ortec PS100 Antidécubitus Heel: Prevention of decubitus ulcers in patients. Ortec HAL100 Hallux Valgus Splint: Correction and slowing the progression of hallux valgus desviaciones reducible phase. Tobi-Stable Ortec TB100, TB110 Ortec Tobi-Stable and TB200 Ortec EMO-Stable: Treatment of severe sprain and repetition, preventative during sports, trauma, stabilizing treatment after prolonged immobilization and post-surgical lesions. Ortec TB300 stabilizing ankle sprains mild preventative during sports, stabilizing after prolonged immobilization. Mediolateral compression pads memory. Ortec TB350 stabilizer with joint: ankle instabilities, immobilization in a position determined after injury, recovery of the functionality of the joint, sports injuries.

Ortec TB600 ankle immobilizing polyethylene: para-articular injuries without fracture, ligament sprains, peroneal, anterior talar and fibular-calcaneal, immobilization in the acute phase of ankle fractures, post-operative.

Instructions for use and placement

Ortec PS100 Antidécubitus Heel: Place the heel of the patient at 90° in the brace. After adjusted by Velcro. Ortec HAL100 Hallux Valgus Splint: Form the strip to the patient's anatomy. Then adjust the brace and close. Tobi-Stable Ortec TB100, TB110 Ortec Tobi-Stable and TB200 Ortec EMO-Stable: Fit the ankle with the foot in neutral position (90°). Adjust the lacing firmly to a heat source. For the covers of orthoses, wash exterior and interior surfaces with a cloth moistened with alcohol.

Quality Certificates
All materials used have been tested, comply with applicable European standards, are high quality and offer maximum safety and comfort during use. Öko-Tex Standard 100: textile eco-label awarded after the completion of a catalog of criteria for analysis to ensure the absence of substances harmful to health. Certify the absence of harmful substances in all the materials that make up a garment.

Notice
To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

Precautions
Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If an allergic reaction should occur remove the product and consult with the prescribing physician. In case of minor annoyances caused by excessive sweating we recommend the use of a tubular cotton. Positioned such that compression is not excessive. Do not

expose the garment to a flame or heat source at very high temperatures. Store at room temperature, preferably in the original packaging. Guarantee within the framework of laws.

Cleaning and maintenance

For pads and adjustment straps: Hand wash with warm water (max. 30 °) and neutral soap. During washing the velcro should be glued together or keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may deteriorate the garment. Do not bleach or dry clean. To lay the garment dry and not exposed directly to a heat source. For the covers of orthoses, wash exterior and interior surfaces with a cloth moistened with alcohol.

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