Ortec

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Dear customer, we appreciate the trust placed in a product EMO. Choosing a brace Osteobrace line you have purchased a quality product and high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or your specialist.

Regulations

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Description and Features

Forearm support strip Ortec SPT100: rigid strap formed by a ring adjustable in each of its ends. Supports the weight of the upper extremity. Lightweight and versatile. Ortec SPT50 padding strip Support: padded strap features adjustable ring on the end. Supports the weight of the upper extremity. Lightweight, comfortable and versatile. Velcro adjustment. Support forearm SPB100 bag: cotton fabric. Supports fixed and the upper extremity in a certain position. Distribute the weight of the whole arm orthoses, including the elbow. Adjust by sliding buckle. With loop for easy wrist rest. Ortec SPB200 Support breathable bag: extremely breathable fabric. Supports fixed and the upper extremity in a certain position. Distribute the weight of the whole arm orthoses, including the elbow.

Velcro adjustment. With loop for easy wrist rest. Ortec SLG100 Emo-Sling: Support locking of the shoulder joint in abduction, rotation and antepulsion. Attaches easily with Velcro to the trunk and adapts to the chosen attitude of the shoulder. Choose the angle of elbow flexion. Distribute the weight of the whole arm orthoses, including the elbow for better support and positioning. Avoid Potential maceration treated arm amput. CHA100 CHA101 Ortec Charpa and bilateral Charpa Ortec: Support locking of the glenohumeral joint. Attaches easily with Velcro to the trunk and adapts to the chosen attitude of the shoulder. Choose the angle of an elbow injury. Ortec HAM100 Brachial Hammock and Hammock Ortec bilateral HAM101: extremely breathable vest that fits over both shoulders. Distribute the weight of the treated arm avoiding the overhead of the cervical area. Attaches easily with Velcro to the trunk. Fits to the desired position of the shoulder and elbow.

Directions

Ortec SPT100 forearm support strap and padded strap SPT50 Ortec Support: shoulder and arm injuries when required support arm control treaty, soft tissue injuries (burns, infections), Support

forearm SPB100 bag breathable bag SPB200 Ortec Support: shoulder and arm injuries when required to control the mobility, positioning and supporting the treated arm, soft tissue injuries (burns infections), rheumatism, post-surgical containment. Ortec SLG100 Emo-Sling and Sling Ortec SLG110 plus: subcapital fractures of the humerus, clavicle fracture, dislocation acromio-clavicular, treatment of shoulder and arm injuries with immobilization of the shoulder after surgery. Ortec Charpa CHA100, CHA101 bilateral Charpa Ortec, Ortec HAM100 Brachial Hammock and Hammock Ortec bilateral HAM101: subcapital fractures of the humerus, clavicle fracture, dislocation acromio-clavicular, treatment of shoulder and arm injuries with immobilization of the shoulder and arm weight bearing , post-surgery.

Instructions for use and placement

The product should be adjusted by the prosthetist or the physician individually for each patient. The correct choice of size is essential for maximum effectiveness of treatment and prolong the life of the garment. Forearm supports, thanks to its design, are easy to install. The patient can be removed and replaced without mobilizing support just the tip, providing rehabilitation exercises and personal hygiene. Ortec SPT100 forearm support strap and padded strap SPT50 Ortec Support: forearm pass (in the area near the elbow) through the ring formed at the end of the tape, the tape passing behind the neck, wrist and finally stand by other end of the tape. Thanks to the pins is adjustable tension bracket, which will give a proper fit. Support forearm SPB100 bag breathable bag SPB200 Ortec Support: enter fully into the forearm support. The elbow should reach the top of the bag and the wrist joint should be within the support. Pass the strap behind your neck and adjust. You can also pass the webbing between the arm and trunk and then over the contra-lateral shoulder. After adjusting. Ortec SLG100 Emo-Sling: forearm was suspended on the support from the tape on the opposite shoulder and through the center ring of the suspension system. Adjust the length of bending the end bracket inward. To keep the tape fastening elbow long with two Velcro is attached to and surrounding the waist side joins the lateral ring of the suspension system. Finally, the tape cut with two Velcro is used to close the holder. Ortec SLG110 Sling plus: the forearm is suspended on the substrate by placing the arm fixed to 90 °. Thread the ribbon "A" behind the neck and on the other shoulder. After spending the same tape "A" below the wrist, fold up on itself and secure with the Velcro end (the weight of the arm and is supported by the sling). Thread the ribbon "B" around the back and pass back and forth between the arm and trunk. Surrounding the injured arm and fix the tape on itself in the back.

To prevent unwanted movement of the forearm on the support we can fix Rafter tape around the wrist. CHA100 CHA101 Ortec Charpa and bilateral Charpa Ortec: placing the stun on the shoulder to be immobilized. Adjust the brace to the trunk connecting the two side velcro to the back of the brace. The front of the brace has to embrace the forearm and pasted, with velcro on the chest to fulfill their support function. Then adjusted

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Notice

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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CE

MADE IN SPAIN

OEKO-TEX®
CONFIDENCE IN TEXTILES
STANDARD 100
2006/AN485 ATEX
Tested for harmful substances.
www.oeko-tex.com/standard100

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