Silicoplant

Dear customer, thank you for the trust placed in EMO product. Choosing this brace you have purchased a quality product and high health medical level. Read these instructions carefully. If in doubt contact your doctor or specialist dealer.

Normative
The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Precautions
Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If you experience any allergic reaction to remove the product and consult with the prescribing physician. Do not expose the garment to a flame or source of heat at very high temperatures. Store at room temperature, preferably in the original container. Warranty within the framework of the laws.

Viscoelastic silicone insoles
Function and Description
They perform a buffer function of the impact of the foot on the ground, providing a sense of comfort and relaxation. Suitable for eke foot such as absence or lack of adequate cushioning generating plant that cause pain plantar plantar algias. Suitable also for people who remain long standing or long marches or exercising sports with repetitive impacts. Silicone pads lower hardness increase discharge action in the most sensitive areas. The Silicoplant Templates™ lined, have a special fabric for improved breathability in people with excessive sweating in addition to raising the capabilities Antibacterial, Anti-odor and Antimicrobial product.

Instructions for use and placement
Place the template inside the shoe, ensuring that the chosen size is right, and that no folds are formed. If necessary cut it out from the front, never on the heel. Wear socks or stockings.

Washing and maintenance
Hand wash with warm water and mild soap. Do not expose to direct heat.

Viscoelastic silicone heel cushions

Function and Description
Achieve a homogeneous distribution of pressure on the heel, reducing the impact on the ground. Suitable for eke foot such as absence or lack of adequate cushioning heel pain plantar plantar generating plant that cause pain. Relax the Achilles tendon, muscles and plantar fascia. Silicone pads lower hardness increase discharge action in the most sensitive areas. The Taloneras Silicoplant™ lined, have a special fabric that improve perspiration in people with excessive sweating in addition to raising the Antibacterial, Anti-odor product Antimicrobial capabilities.

Instructions for use and placement
Place the heel within the shoe proving that size is adequate, without leaving slack and without folds forming. Wear socks or stockings.

Washing and maintenance
Hand wash with warm water and mild soap. Do not expose to direct heat.
Quality certificates
All materials used have been tested, comply with applicable European standards are of high quality and offer maximum safety and comfort during use, Öko-Tex Standard 100: Textile Eco-label awarded after the completion of a catalog of criteria for analysis that guarantees the absence of substances harmful to health. Certifies the absence of harmful substances in all materials with which is made of a garment.

Warranty
Orthopédic Medical Specialties S.L. guarantees all its products as long as they have not been tampered with or altered and always within the framework and terms of laws. Products whose characteristics have been altered due to misuse are not included within this warranty.

Notice
To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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