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AFO FUNCTIONAL AND PLASTERING

The aim of these orthoses is to get through the early onset of the injured limb joint mobility, restoration or the functionality of the member and therefore the incorporation of the patient to activities of daily life in the shortest time possible no residual deformity and avoiding the problems associated with immobilization (disuse muscle atrophy, joint stiffness, osteoporosis, etc.). Wherever possible the use of these orthoses must be accompanied by appropriate rehabilitation treatment, a training in the use of the brace and complete information on the objectives we seek to use it.

Place the injured limb within the splint brace and adjust the straps closed without exerting excessive pressure but by a sure grip.

If at some point the splint rubs, prevents movement of adjacent joints or simply not convenient they may be trimmed edges and then smoothing them warm with an industrial blower and shape the plastic, always helping with heat-resistant gloves. Once adjusted the splint tape leftovers can be cut for comfort.

BRACE OF HUMERUS BIVALVE

Functional treatment of humeral bone fractures, average state of cure, treatments associated with fixation of alignment, delayed union, after the removal of osteosynthesis material.

To immobilize the fracture site while maintaining mobility of joints.

Formed by two valves, the posterior leaflet covers the deltoid laterally, conforming to the acromion, extending about two thirds of the arm later. The anterior leaflet has the same shape, but covering two thirds of the anterior arm then arches distally to facilitate flexion of the elbow.

Sizes: (length / contour arm in cm.)
S (28/25) M (32/30) L (37/35) XL (37 /> 35)

Side: Right and Left

FBR20 BRACE ARTICLES

Functional treatment of upper limb fractures, average state of cure, treatments associated with fixation of alignment, delayed union, after the removal of osteosynthesis material. To immobilize the fracture site while maintaining mobility of joints.

Formed by a valve at double-piece humeral and forearm level, which are adjusted and fixed by means of tapes. Both sides are joined by a couple of joints that can stabilize the motion. The position of the junctions are regulated via a slide on the blanks in the longitudinal and through a slotted boxes in thermoplastic. This form of regulation provides a simple adjustment of the joints even when placed on the patient.

Sizes: arm length / contour arm / long arm in cm.
S 28-33 / 25-30 / 22-28
L 32-37 / 30-37 / 22-28
Side: Right and Left

FAB20 FOREARM MINIBRACE/FAB10 FOREARM MINIBRACE WITH TILE

Functional treatment of fractures of the ulna and / or radio, media statements healing treatments associated with fixation of alignment, consolidation delay, after the removal of osteosynthesis material. To immobilize the fracture site while maintaining the joint mobility

Formed by two valves, which cover the forearm, conforming to the elbow joint. The tile version incorporates an extension of the thermoplastic in the distal-dorsal limits the dorsiflexion of the hand, and even with the use of a fastener strip can be achieved by immobilizing the wrist joint.

Sizes: (forearm contour cm.)

S (22) M (25) L (28)

Side: Right and Left

CUPS WITH ANKLE JOINT FTB10 CONTROLLED / FTB35 FREE

Nullifies the effect "impact" on the heel strike phase and prevents the cast or formed spat could fall due to their weight. The bowl whose function is to hold and locate the foot, is adjusted by means of a tape. The two lateral pins are attached to the brace by means of rivets, or functional gypsum plaster bandages.

Models: controlled Ankle / Ankle free

Sizes: L and XL

FPR10 CUPS WITH TIBIAL GAITERS

Functional treatment of fractures of the tibia and ankle, media statements healing treatments associated with fixation of alignment, delayed union, after the removal of osteosynthesis material.

Made in two parts, the posterior leaflet contains all the calf with a slight depression in the area of the popliteal fossa. The anterior leaflet, completely encloses the leg, has shaped the patellar support. The bowl (controlled) with the product, properly positions the heel with the spat through two lateral pins, which are riveted to the spat in the desired position.

Sizes: (outline calf in cm.)
S (29-32) M (33-36) L (37-40) XL (41-44)

Side: Right / Left.

FML10 FEMUR LEGGUARDS

Suitable for the functional treatment of diaphyseal fractures of the femur. Shown in middle stages of healing, during the consolidation of periosteal callus. After acute phase of fractures and treatments associated with osteosynthesis

Assemblage consisting of a thermoplastic ischial support, which fits into the more proximal lower limb hug the contours of the thigh with tape closure. Sizes: (outline proximal / height / ischium-adductors in cm.)
S (46/25/7) M (50/28/8) L (54/28/9) XL (58/30/10, 5) XXL (61/30/11)

Side: Right / Left.

FTB50 DOWNLOAD CALCANEUS

Functional treatment after the acute phase in fractures of the talus and calcaneus. Also after osteosynthesis in fractures of the tibia, in bi-malleolar fractures,

or other fractures or necrosis at the foot. Consisting of a PTB socket type supported effectively allows you to download the bottom of the leg. It should cover the condyles by the side wings, the knee being housed inside the brace. Its shape allows the lift at the popliteal region and support mainly subtrotuliano. The height of the brace is regulated by the tensile side of the hardware on the move in the same on both sides. The forefoot should embrace the fastening tape, so that the foot is in suspension, allowing the cushioning during walking, it is necessary to use up that balance the contralateral limb.

Sizes (calf contour cm.)

S (29-32) M (33-36) L (37-40) XL (41-44)

Side: Right / Left.

ELBOW AND KNEE JOINTS

Biomechanical design that allows joint mobility across the bow flexo-extensor of the elbow joint or knee. Formed with a joint together by monocentric or polycentric to model and can function freely and gradually controlled or limited by model.

Limits the potential pathological movements (middle-side) of the joint and allows limits or blocks the controlled flexion and extension of the arm or knee. Slotted mounting plates allow us to adjust and regulate the position of the joints and adapt both orthoses as functional casts

FRD20 polycentric knee joint

Allows full mobility of flexo-extensor bow the knee. A pin joint allows us to block the extension.

FRD50 polycentric knee joint flexion and extension limited

Allows full or limited mobility of flexo-extensor bow the knee. The pin-regulation allows us to limit the arc in flexion or extension depending on the position of the same, then the joint is protected by a cover.

Articulación monocentric FCD10 elbow
Articulación monocentric with the possibility of limiting the extension to 15° and 45° flexion and 75°

FCD20 elbow joint polycentric

Polycentric joint arc of elbow flexion and extensor. You can lock in extension and 35° and 70° of flexion, or limit the extent of 75° or 40°.

FCD30 progressive elbow joint

Articulación monocentric, progressive control of the flexion and extension, through a system of pins of a micro-metric control the progression of the movement.

FCD35 free elbow joint

Articulación monocentric to stabilize the shoulder to move freely, without restrictions throughout the flexo-extensor arch.

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Normative

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Precautions

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional.

Notice

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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