Rizogrip

Dear Customer, We appreciate the confidence placed on a product EMO. Choosing a brace Rizogrip brand you have purchased a quality product and high level medical doctor. Read these instructions carefully. If you have any questions please contact your doctor or your specialist.

REGULATIONS

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND FEATURES

MQ705 Rizogrip stabilizing the thumb. Its anatomical shape stabilizes and immobilizes the joint trapezium metacarpal and metacarpophalangeal. Technologically Advanced: manufacture by micro - fusion avoids overpressure and scratches, absence of seams. Light: made with both maximum strength and minimum thickness. Incorporates a malleable aluminum ferrule adaptable to the morphology of the patient. Standing with one hand: adjustment strap that closes in on itself for easy adjustment placement. Perfect: by webbing through buckle. Breathable fabric and foamed. Bilateral. S (14-18 cm), L (18-22 cm)

INDICATIONS

Trapezometacarpal Osteoarthritis of finger 1° (rhizarthrosis), acute and chronic inflammation of the joint, tendinitis, injury of the skier.

INSTRUCTIONS FOR USE AND PLACEMENT

The product should be adapted by the prosthetist or by the physician individually for each patient. The correct choice of size is essential to achieve maximum effectiveness of treatment and prolong the life of the garment. Loosen the straps and fasten on themselves. Check the shape of the splint and adapt to the morphology of the patient's hand. Enter the injured hand and adjust the straps around your wrist and thumb. The patient can get on and off the cuff without mobilizing Just The Tip, facilitating rehabilitation exercises and personal hygiene.

PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult

a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If you experience any allergic reaction to remove the product and consult with the prescribing physician. Do not expose the garment to a flame or source of heat at very high temperatures. Store at room temperature, preferably in the original container. Warranty within the framework of the laws.

CLEANING AND MAINTENANCE

Hand wash with warm water (max. 30 °) and neutral soap. During washing, if the garment has Velcro, they must stick together or keep them closed. Dry at room temperature. Do not iron the garment shop. Contact with ointments, creams, alcohol and solvents can damage the garment.


QUALITY CERTIFICATES

All the materials used have been tested, comply with the applicable European regulations, are of high quality and offer maximum safety and comfort during use. Öko-Tex Standard 100 Certificate: Textile ecological label awarded after the completion of a catalog of analysis criteria that guarantees the absence of substances harmful to health. It certifies the absence of harmful substances in all the materials with which a garment is made.

NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

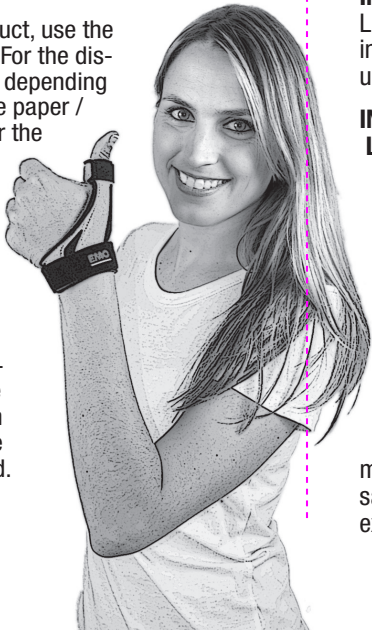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