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1. INTRODUCTION

Orthos XXI products are equipped with equipment designed to serve the sick, the disabled and the elderly, with multiple disabilities, so its use should be advised by a health professional or distributor who knows the product and the purposes for which it is intended, as well as this user's manual. Orthos XXI is not responsible for cases where the device is not suitable for the user's disability.

Orthos XXI has an ISO 9001 certificate, which validates the quality of products during their manufacture. The vast experience and manufacturing technology allows us to adjust the products to the different physical disabilities of the end customer. Walking aids are Class I medical devices according to EU regulation 2017/745.

Proper use, in accordance with these instructions, guarantees safe use for the user. Therefore, we recommend reading this manual and applying the safety recommendations. Orthos XXI reserves the right to make changes to the device due to technical improvements, as well as to modify the instruction manual at any time. The information contained herein may vary slightly from the present device.

Thank you for choosing an Orthos XXI product.

2. SAFETY INSTRUCTIONS

Before using the device, make sure that the dimensions of the walking aid are suitable for the user.

Read these instructions before using the device to prevent damage to yourself or the device!

2.1. General safety instructions

- Always use the device in accordance with the instructions in this manual.
- Use the device only for its intended purpose. Pay attention to the recommended weight limit.
- Regularly inspect the product to ensure that no damage has been done to the device.

- Regularly clean the product, ensuring that the patient contact surface is free of any cleaning residue. See cleaning instructions.
- Be aware that, as an assistant, walking aids must not be used improperly (knocking on doors, throwing objects, strong impacts or shocks, etc.).
- Replace tips whenever they show wear or deterioration.
- Do not use walking aids away from the body, or poorly supported, to avoid accidents due to imbalance or falls.
- Pay attention to smooth, wet and/or slippery floors, which could cause the user to become unbalanced and/or fall.
- Pay attention to the contact of the walking aid with chemical products that lead to premature corrosion of the tip.
- Uneven floors (parallel, grids on sidewalks, small unevennesses, etc.) can cause the tip to become stuck or obstruct its movement and cause imbalance and/or fall, so the walking aids must be well raised and well supported to pass over stones or uneven sidewalks.
- Soft surfaces are highly dangerous and can bury the device and cause the user to become unbalanced.

2.2. Weight limit

- The maximum user weight must not be exceeded. You can find the maximum user weight in section 7 TECHNICAL SPECIFICATIONS.

3. PURPOSE

Walking aids (elbow crutches, canes, axillary crutch and pyramids) are walking aids for external use, which replace all or part of a lower limb of the human body and are used by people who are temporarily or permanently incapacitated, sick, disabled or elderly people on their journeys on foot.

4. GENERAL FEATURES

The walking aids are made of aluminum tubes, with a thermoplastic handle, adjustable in height by metal pins, with rubber tips at the ends in contact with the ground. The cane has a PP handle. The elbow crutch has an articulated or fixed thermoplastic clamp. The axillary crutch has a wooden handle and a polyurethane armpit. The pyramid has a PVC handle.

For a better fit to the user, several models of each type of walking aid are available.

• Elbow Crutches

- C1 Integral
- C8 Integral
- C7
- C1 Traditional
- C8 Traditional
- C4 Junior
- Forearm

• Canes

- Aluminium
- Wood
- Blind

• Pyramids

- Quadripod P2
- Tripod P1
- Quadripod P4
- Tripod P3
- Quadripod P6
- Tripod P5
- Quadripod P10

• Axillary Crutch M1

5. REGULATIONS

The elbow crutches must be adapted so that the arm and hand are well supported, either on the wrist or on the clamp.

Axillary crutches should be adapted so that the hand rests well on the wrist, and the armpit rests on the crutch.

Canes and pyramids must be adjusted so that the arm is well extended in order to guarantee good support to the forearm and shoulder and, through these, to the entire bone structure, safeguarding degenerative states due to incorrect postures.

Use the pyramid so that the supports are outwards (not on the user's side.) Pyramids must not be used on stairs.

The position of the walking aids should be as vertical as possible, and their movement should be carried out in small steps following the movement of the body in order to keep the body always well supported.

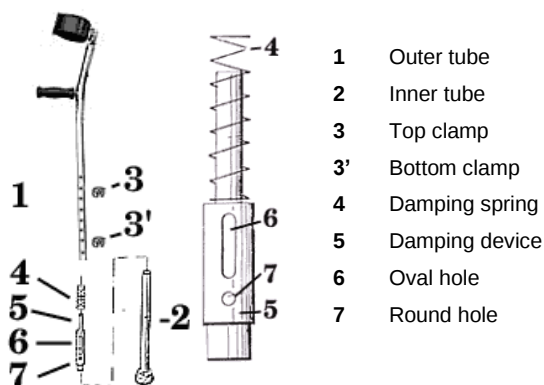
5.1. Height adjustment

To adjust the height of the device that includes a damping system, you must:

1. Insert the clamp (3) in the appropriate hole to adjust the height of the device;
2. Insert the inner tube with the spring placed as shown in the figure;
3. Place the lower clamp (3') two holes below the first clamp:
 - If it goes through the oval hole (6), the damping effect is obtained;
 - If it goes through the round hole (7), the damping is canceled.

To adjust the height of the device without a damping system, you must:

1. Remove the clamp (3);
2. Adjust the elbow crutch to the desired height;
3. Replace the clamp (3).

5.2. Tips replacement

Whenever the tip shows wear, it must be replaced.

To do this, you must:

- 1st Remove the used tip;
- 2nd Insert the new tip.

6. MAINTENANCE AND/OR REPAIR INSTRUCTIONS

6.1. Cleaning/disinfection care

The device should be cleaned regularly. We recommend the following:

- Wash the device with a cloth dampened with a neutral cleaning product diluted in water;
- The equipment must be dried using a soft textured cloth;
- No abrasives or strong acids or bases should be used;
- The equipment must be disinfected with alcohol and derivatives, whenever they are in diluted concentrations
- Thermoplastic constituents should be inspected regularly and replaced when they show wear or fracture. The tuners (inner tube) must be regularly lubricated with vaseline.

6.2. User maintenance

All Orthos XXI products leave the factory inspected and tuned for immediate use. Its continuous use requires routine tuning and maintenance, especially if the original assemblies have been altered.

6.3. Maintenance and repair by the manufacturer

Regular maintenance allows the identification and replacement of defective and worn parts, which improves the daily operation of your device.

In case the manufacturer's intervention is necessary for maintenance and repair, contact your distributor.

Whenever you send the device for repair and/or maintenance, you should report any anomalies you have detected. For example,

- Strange noises/vibrations on wheels or other components;
- Loose pieces;

- Damage or cracking.

The table below shows the regular inspections you should carry out on your device to continue to ensure its safety and efficiency.

	In delivery	Whenever necessary	Weekly	Monthly
General • Check that all parts are well assembled • Check if there is any noise			✓ ✓	
Structure • Check that the structure is not damaged or cracked • Check the wear of the tips	✓	✓	✓	✓
Cleaning • Carefully clean and/or disinfect all parts and components		✓		

7. TECHNICAL SPECIFICATIONS

Walking aid		Maximum user height * (mm)	Maximum user weight * (kg)	Distance leg/grip (mm)	Distance grip/rim (mm)	Arm support angle (°)
Elbow crutch	C4 Junior	1600	100	620 – 830	210	24,2
	C1/C8 Traditional	1800	100	710 – 960	210 – 310	24,2
	C7	1800	140	710 – 960	210	24,2
	C1/C8 Integral	1800	150	710 – 960	210	24,2
	Forearm	1800	100	940 – 1200	-	-
Axillary crutch		1800	100	760 – 980	380	-
Cane		1800	100	810 – 960	-	-
Pyramid	P1/P2	1800	100	800 – 950	-	-
	P3/P4/P6	1800	100	570 – 710	210	24,2
	P5	1800	100	590 – 740	210	24,2
	P10	1800	100	745 - 955	-	-

According to standard ISO 11334 Assistive products for walking manipulated by one arm – Requirements and tests methods

* For different tabulated height or weight, the devices can be executed by special request to the manufacturer

8. MANUFACTURER'S WARRANTY

The structure of the devices included in this manual is covered with a 24-month warranty from the date of purchase.

During this period, Orthos XXI is responsible for replacing or repairing, free of charge, parts that have a manufacturing defect, if one of our specialized technicians verifies that the damage was not caused by improper use.

The following points are not covered by the warranty:

- **Inappropriate use**
- **Modifications without prior authorization**
- **Incorrect storage**
- **Poor conservation**
- **Normal wear and tear**
- **Malfunctions due to lack of routine maintenance**

The products included in this manual cannot be modified. Any modification, including the assembly of unrecognized spare parts or accessories, may cause insecurity in the product and invalidate any safety and warranty mark.

Orthos XXI will not accept any liability for modified products or for any damage or injury arising from these modifications.

The lack of care with ordinary and extraordinary maintenance operations releases the manufacturer from any responsibility.

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At **Orthos Walking Aids** you will always find **life long** crutches and walkers, **more than 30 years in our hands...**

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